Anuform®

Instruction Manual – GB

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Manual de Instrucciones – ES

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Manuale d'Istruzioni – IT

Handleiding – NL

Brukerveiledning – NOR

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inner strength, outer confidence

Specifications

Materials:

Probe body: High Impact Polystyrene

Electrodes: Medical Grade Stainless Steel

Cables: PVC over copper tinsel conductors

Weight: 23.5g, Cable Length: 30cm

Connectors: 2 x 2mm diameter sockets

Electrode Surface Area: 3cm² per electrode

'In Use' product lifetime: 6 months

Contents

2 - 3	Indications for Use - GB
4 - 5	Indications - FR
6 - 7	Anzeigen für Gebrauch - DE
8 - 9	Indicaciones de Uso - ES
10 - 11	Indicações de utilização - PORT
12 - 13	Istruzioni d'uso - IT
14 - 15	Gebruiksaanwijzing - NL
16 - 17	Indikasjoner - NOR
18 - 19	Käyttöohje - FIN
20 - 21	用途 - SC

Indications for Use:

For use as an intra-anal probe in males and females and a small vaginal probe in females. For use only with muscle stimulation and EMG biofeedback equipment approved to EN60601-01 in the treatment of 'Stress', 'Urge' and 'Mixed' incontinence and faecal incontinence.

FOR SINGLE PATIENT USE – may be re-used by the same patient.

FOR SINGLE ORIFICE USE – not for both vaginal and anal use.

Use as directed by your medical adviser.

Contraindications & Precautions:**DO NOT USE:**

With electrical stimulation if fitted with a cardiac pacemaker • If pregnant
• If the package seal is not intact • If an infection or tissue damage or recent surgical scar is present • Vaginally during menstruation • With electrical stimulation in patients with a history of carcinoma at the site of stimulation
• If an irritation or discomfort occurs and seek medical guidance.

CAUTION IS ADVISED:

When using electrical stimulation on patients with altered or reduced tissue sensation • During electrical stimulation if the user has a history of epilepsy - seek informed medical guidance • When using the ANUFORM® with equipment capable of delivering outputs in excess of 2mA/cm². If in doubt contact the equipment manufacturer or your medical adviser.

Cleaning:

- Wash ANUFORM® in warm soapy water.
- Rinse in clean water and dry.
- Repeat after each use and dry thoroughly before storing in its plastic bag.
- DO NOT use boiling water.

Disposal:

Clean thoroughly and dispose of according to local regulations.

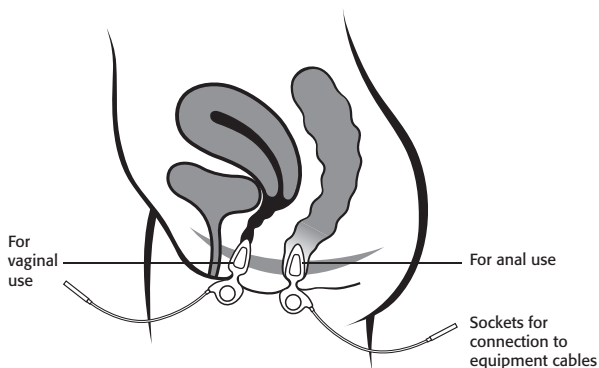
Contains nickel:

May cause irritation or allergic reaction following prolonged contact.
Always follow the user instructions.

Indications for Use:

IMPORTANT:

- Before first and subsequent use wash and dry as directed.
- Apply light coating of KY Jelly or similar lubricating gel to tip and metal surfaces of probe to aid insertion and provide good electrode conductivity.
- **Anal use:** Insert into anus while 'bearing down' as if passing stool. 'Ring' part of ANUFORM® should remain external to the anus at all times.
- **Vaginal Use:** Insert into vagina in an 'East/West' position, i.e. with the metal electrode surfaces facing towards the hips. Insert until body of ANUFORM® is within the vaginal vault. 'Ring' part of probe should remain externally within labia.
- Ensure equipment is turned off before connecting to probe cables.
- Use as directed by your medical adviser. After use, turn off equipment, disconnect cables. Do not pull on cables to remove probe.
- Wash, dry and store as instructed on previous page.

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Patterson Medical Ltd. Nunn Brook Road,