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Dr. Arabin GmbH & Co KG im FEZ  
Alfred-Herrhausen-Str. 44, 58455 Witten  
Tel: (49) 0 2302 1892 14 Fax: (49) 0 2302 1892 16  
[www.dr-arabin.de](http://www.dr-arabin.de) [info@dr-arabin.de](mailto:info@dr-arabin.de)

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D-CA+ASQ

## Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ  
Alfred-Herrhausen-Str. 44, 58455 Witten  
Tel: (49) 0 2302 1892 14 Fax: (49) 0 2302 1892 16  
[www.dr-arabin.de](http://www.dr-arabin.de) [info@dr-arabin.de](mailto:info@dr-arabin.de)

## Instructies

Dr. Arabin GmbH & Co KG im FEZ  
Alfred-Herrhausen-Str. 44, 58455 Witten  
Tel: (49) 0 2302 1892 14 Fax: (49) 0 2302 1892 16  
[www.dr-arabin.de](http://www.dr-arabin.de) [info@dr-arabin.de](mailto:info@dr-arabin.de)

### Cerclage Pessary

#### (Typ A without perforations / Typ ASQ perforated)

**Application:** The cerclage pessary consists of flexible comfortable silicone. Like all silicone pessaries it is elastic, it can be folded and thus be inserted without pain. The cerclage pessary is designed for the treatment of pregnant women, to support the cervix mainly in patients with additional complaints of prolapse (painful "downwards" pressure during standing and walking), pregnant women who are exposed to physical strain (e.g. standing for a long time), increased intrauterine pressure, e.g. multiple pregnancies or ultrasound signs of an incompetent cervix.

Before insertion bacterial swabs should be performed to exclude bacterial colonization. The treatment should start before the cervix might be effaced. If used on a therapeutical basis it is meanwhile indicated when transvaginal sonography reveals cervical shortening and/or a dilatation of the internal os, usually between 15 and 20 weeks. The pessary is not meant to close the cervix but rather to support it and to shift the cervix towards the sacrum.

In case of increased cervical/vaginal discharge perforated pessaries (ASQ) allow a better passage of fluids. In a randomized study it was demonstrated that increased discharge is not combined with a change of the bacterial pattern, infection.

**Sizes:** Cerclage pessaries differ in their outer diameter (65mm to 70mm) as well as in the height of the curvature (each 17mm, 21mm, 25mm, 30mm). The inner diameter for all models is either 32mm or 35mm. The higher models are preferred under more severe conditions.

**Use:** The cerclage pessary is inserted in a recumbent position by the obstetrician. It is vital that the curvature of the pessary shows upwards so that the larger diameter is supported by the pelvic floor. The pessary can be folded and then be placed into the vaginal fornix. Thereby the cervix should be located within the upper ring diameter. The obstetrician may control whether the cervix is not surrounded too firmly. After the insertion the patient is asked to get up. Normally patients indicate "relief of prolapse complaints" immediately after placement. The insertion of the pessary can be facilitated by spreading a gliding compound, preferably antibiotic creams not destroying the natural flora.

The pessary should be removed before delivery at around 37 weeks in asymptomatic patients. In cases of increased discharge it may be changed and cleaned in between, but it is not mandatory if manipulations have to be better avoided. In all patients with premature rupture of membranes, vaginal blood loss and severe painful contractions the pessary should be removed to avoid cervical lesions or ascending infection. Vaginal sonographic examinations can help to identify the cervical shape and length but the visualization of transvaginal sonography may be impaired by the pessary. The position of the pessary can easily be controlled manually during routine controls.

**Side Effects/Complications:** The right position of the pessary (with the curvature and the smaller diameter upwards) is the key in the success of treatment. Otherwise the supporting effect is insufficient. There might be some increase of abacterial discharge. However, the composition of the vaginal flora will not be substantially altered.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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