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D-SP+SSP

Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

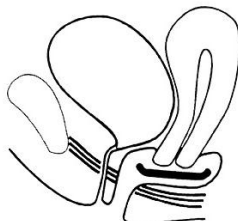
Bowl or Sieve Bowl Pessary

Application: The bowl or sieve bowl pessaries consist of flexible silicone. Therefore they are elastic and can be easily folded and can be inserted and removed without causing pain. **Preferably mild forms of pelvic relaxation and prolapse are treated by the bowl pessary.**

In patients with a cystocele the pessary supports both the medial and the lateral defect. The contact to the pelvic floor (Figure) also increases adhesion. However, an intact pelvic floor is required for this pessary.

Sizes: Various sizes of the bowl pessary can be indicated, ranging from a minimum of 55mm to a maximum of 95mm in diameter. The smallest pessary that just holds should be inserted. The pessary can also be used preoperatively to test the effect of a suspension operation (to exclude possibly harmful effects) and to increase tissue circulation and epithelium formation by the additional estrogen effect.

Use: The physician adapts the pessary on the first examination. The right application should be tested by provoking stress conditions such as coughing, pressing and movements. Coating with an estrogen cream facilitates the insertion. After the test, the position of the pessary should be checked again. **The pessary can be changed by the patient herself, i.e. she has to remove it in the evening and to insert it in the morning. Thereby the pessary should be covered with estriol cream** (short adhesion on the receptor). Since one needs a relatively large amount of cream (large surface) to reach a gliding effect, we recommend to use a low dispensed cream of 0.1 mg estriol / 1 g cream, in rare cases of up to 0.5 mg estriol / 1 g cream. The estriol may improve tissue circulation and epithelium formation.

The change of the pessary may be performed in a standing position whereby the patient may put one foot on a stool or the edge of a bed; if this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying. The patient should be trained how to insert the pessary (e.g. first to press it and then push it to the upper part of the vagina) and how to remove it (surrounding of the ring with her forefinger, edging and loosening the pessary). However, older patients may prefer a changing by the physician or a nurse at intervals of several weeks.

Side Effects/Complications: The therapy with the bowl pessary has the aim –possibly in combination with additional measures - to reduce the patient's problems due to the prolapse. If the patient changes the pessary daily, by using estriol crème after the menopause but even if the pessary is only changed by physicians or nurses, complications such as infections or tissue lacerations should be avoided. If the prolapse is insufficiently cured or if various sizes do not stay in their position, one has to change to a cube pessary and local estrogen therapy. Contraindications for estriol creams should be considered (e.g. pregnancy, lactation period, estrogen-dependent tumors). If the patient is not able to change the pessary by herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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