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D - UP

Instructions for Pessary Treatment

Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Instructies

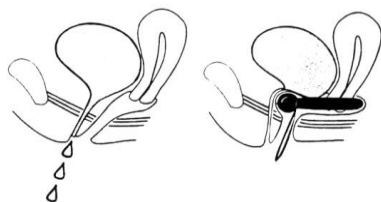
Dr. Arabin GmbH & Co KG im FEZ
Alfred-Herrhausen-Str. 44, 58455 Witten
Tel: (49) 0 2302 189214 Fax: (49) 0 2302 189216
www.dr-arabin.de info@dr-arabin.de

Urethra Pessary

Application: The urethra pessary consists of flexible silicone with a steel spring filling which keeps its elastic form. The pessary is mainly indicated in patients with **stress incontinence possibly combined with descensus but with still intact pelvic floor**. The pessary might prevent the development of further stress incontinence by the thickening of the pessary which is supposed to support the transition between bladder and urethra and to avoid an opening of the upper urethra under stress situations like coughing or movements (Figure). A shifting can also avoid seeping of urine into the urethra, which may even have a positive effect on urge or a **mixture** of stress and urge incontinence.

Sizes: The diameters of the pessaries range from a minimum of 45 to a maximum of 100 mm. The size which achieves the best effect to assure continence when coughing in the standing position and still allows for micturition should be chosen. One might have to change to a larger pessary after 1 or 2 months, due to a relaxation of the vaginal tissue. When inserting the pessary, the ring will be pushed through the vaginal vault upwards so that the thickening lifts the transition between bladder and urethra (picture).

Use: The physician adapts the pessary on the first examination with a filled bladder on the examination chair. Subsequently, the patient is asked to cough, press and to move and the loss of urine is registered and the position of the pessary checked again. With an adequate position of the pessary, also a severe stress incontinence should be redressed. Before the insertion, the pessary is covered with estrogen cream. We thereby recommend a mixture of 1 mg estriol/1g cream (short adhesion to receptor). It is recommended that the patient learns to change the device by herself, usually she might remove it in the evening and insert it in the morning or use it only for certain conditions such as sports. The coating with estrogen cream facilitates the insertion and improves the local blood circulation as well as the formation of epithelium and connective tissue. The change of the pessary can be performed in a standing position whereby the patient may put one foot on a stool or the edge of a bed; if this is too difficult, it may be sufficient to insert the pessary while spreading the legs, if necessary while leaning against a wall or while lying. The patient should be trained how to insert the pessary (e.g. first to press the ring part and then push it to the upper part of the vagina whereby the thickening lies under the symphysis) and how to remove it (surrounding of the ring with her forefinger, edging and loosening of the pessary). The fixation of a thread on both sides of the thickening, which can be adapted individually may facilitate the handling and avoid a dislocation of the thickening under the bladder. However, older patients may prefer to be assisted by the physician or a nurse at regular intervals. Under these conditions, one should insert an estriol cream or ovulum once a week. The pessary can be used in combination with local estrogens before a planned vaginal operation to improve circulation of the vaginal tissue.

Side Effects/Complications: The therapy with this pessary is meant - in combination with additional measures (physiotherapy, estrogens) - to cure or at least to reduce the patient's problems of incontinence. The daily change avoids complications like infections, bleedings or even ulcers. Stress incontinence does not need any therapy during the night anyway. If any problems with incontinence remain, the diagnosis and choice of the pessary have to be reconsidered. It is possible that the thickening cannot get upwards because of insufficient levator muscles or an obstructed vaginal wall. Under these conditions, one has to change to a cube pessary and local estriol therapy - at least temporarily. Contraindications for estriol creams should be considered (e.g. pregnancy, lactation period, estrogen-dependent tumors). If the patient is not able to change the pessary herself, it may be advisable to integrate a nurse or a member of the family into the procedures.

The product must not be used by other patients and be stored by room temperature. **The pessary may be cleaned under warm running water without using any disinfectant.** The material can be cleaned and disinfected preferably with aldehyd-containing solutions (Gigasept FF). The pessary can also be sterilized by steam (134°C /3,8 bar) or hot air (250°C).

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