P E R I F I T

- 1 English
- 5 Français
- 9 Deutsch
- 13 Español

P E R I F I T

English user guide

Thank you for purchasing Perifit. We hope that Perifit will bring you quick and enduring results.

Here are the first steps to get started using your Perifit:

1. Download the Perifit app on your smartphone or tablet (look for Perifit in the App Store or Google Play).
2. Activate Bluetooth on your phone, but don't try to connect it to Perifit within your Bluetooth settings.
3. Launch the Perifit app and follow the detailed in-app instructions on how to use Perifit.

You can reach our customer support at any time by using the email button inside the app, we would be more than happy to help you.

How to take care of your Perifit

Clean your Perifit before and after every use with warm, soapy water.

Remember to rinse thoroughly. It is important that your Perifit is completely dry before you store it.

Store your Perifit in a clean and dry environment.

DO NOT:

- Wash in boiling water
- Sterilize
- Use cleaning agents
- Microwave
- Clean in a washing machine
- Leave it trapped under any object
- Use your Perifit during a bath
- Silicone based lubricant

AVOID:

- Extreme heat
- Extreme cold
- Sustained pressure

Safety instructions

Do not use your Perifit if:

- You are pregnant.
- You have given birth within the last 6 weeks.
- You experience persistent pain when inserting or using Perifit, even with water-based lubricant.
- You are suffering from a severe condition affecting your pelvic floor (severe prolapse, severe vaginismus, etc.) and have not already spoken to a pelvic floor specialist.

If you feel any pain during or after use of your Perifit, contact a qualified specialist.

Perifit is a muscle strengthener. It must not be used to diagnose a medical condition. If you are not sure whether you should be using your Perifit, please contact a women's health professional.

Perifit is a single patient device, intended to be operated by adult female patients, which do not need to have any formal education, but it is expected that they know how to read and use a smartphone.

Do not leave your Perifit in for more than one hour at a time.

For further information, please refer to the FAQs in the Perifit app or contact us via the email button in the Perifit app.

Guarantee

Perifit is made from medical-grade silicone and is backed by a limited manufacturer's guarantee against any defects in materials and/or workmanship for 2 years from the date of purchase.

Defects discovered within the guarantee period should be reported to X6 Innovations.

X6 Innovations reserves the right to exercise its discretion in determining whether or not to replace a defective Perifit free of charge.

This guarantee does not cover cosmetic deterioration or damage caused by general wear and tear, physical or natural destruction, accident, misuse, neglect or other external causes.

Any attempt to take the Perifit apart will invalidate this guarantee. Any guarantee claims must be supported with reasonable evidence, including proof of date of purchase, alongside these guarantee conditions. X6 Innovations must be notified of any defective Perifit.

We will then provide you with a Return Merchandise Authorisation number, which must be displayed on the outside packaging. In the event that X6 Innovations decides to replace your Perifit, any replacement Perifit will not extend the life of this guarantee.

DISCLAIMER:

Users of Perifit do so at their own risk. Neither X6 Innovations nor its retailers assume any responsibility or liability associated with or for the use of this product.

X6 Innovations reserves the right to revise and/or update this manual without being obligated to notify any users.

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Icons explanation

This packaging contains a printed user guided (as you can see).

This is a medical device certified CE class 1.

This medical device has no electrical contact with the patient.

This device should be keep between 15°C and 60°C (59°F and 140°F).

This device should be keep between 15% and 80% humidity.

Do not throw away.

Presence of non-ionizing electromagnetic radiation.

RoHS

This device is compliant with the RoHS directive about hazardous substances.

IP27

Perifit can be temporarily submerged in water.