Novuqare® MAPLe® probe

Manufacturer:

Novuqare Pelvic Health BV

Kievitsven 42
5249 JJ Rosmalen
The Netherlands
T: +31 (0)73 782 01 18
F: +31 (0)73 782 01 11
cs@novuqare.com
www.novuqare.com

0344

See bottom probe:
YY WK


IP25

See manufacturing
date + **5 years**

LOT See bottom
probe: **YY WK**

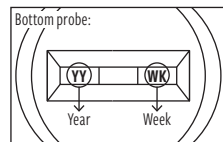v1.7

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ENGLISH - INSTRUCTIONS FOR USE

The MAPLe probe is suitable for anal and vaginal Electromyography (EMG) measurements and electro stimulation. The MAPLe probe may solely be used i.c.w. the MAPLe system; max. stimulation current 29mA - max. voltage 35V.

Instructions for Use

Store the MAPLe probe at room temperature in a dark and dry location. The MAPLe probe is strictly personal and should be cleaned before and after use with the following protocol:

- Wipe cable and probe with a tissue and dispose the tissue.
- Rinse the probe and cable (still attached to each other) under lukewarm water from the tap.
- Dry the material with a soft cloth or a soft, non-fluffy tissue.
- Disinfect probe and cable with a gauze which has been well wetted with alcohol 70%. Maintain a contact time of at least 30 seconds.
- Remove the probe from the cable with the gauze pad, put the probe aside to dry.

- With regard to the probe cable: Disinfect the rim, ring and inside of the plug at the side of the patient with a gauze pad well-moistened with alcohol 70%. Be sure no alcohol seeps into the black connector.
- Maintain a contact time of at least 30 seconds. Air dry the cable.

Contraindications

Do not use the MAPLe probe for electro stimulation in the presence of confirmed or suspected: Colo-rectal or urogenital cancer / Pregnancy / Implanted electrical devices (e.g. cardiac pacemaker); only after consulting a specialist / High fever

Warnings

- Keep the inner part of the probe dry.
- The probe cannot be sterilized.
- Simultaneous connection of a patient to the MAPLe system and high frequency surgical equipment may result in burns at the site of the stimulator electrodes and possible damage to the stimulator.
- Operation of the MAPLe system in close proximity (<4 meters) to shortwave or microwave therapy equipment may produce instability to the stimulator output. Please maintain a distance of at least 4 meters.
- Operation of the MAPLe system in close proximity to cellphones may produce instability to the MAPLe system. To achieve the best results it is highly advised to switch off mobile phones when using the MAPLe system.
- Do not use the MAPLe probe in case of damaged electrodes.
- Current densities exceeding 2 mA/cm² may require the special attention of the operator.

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