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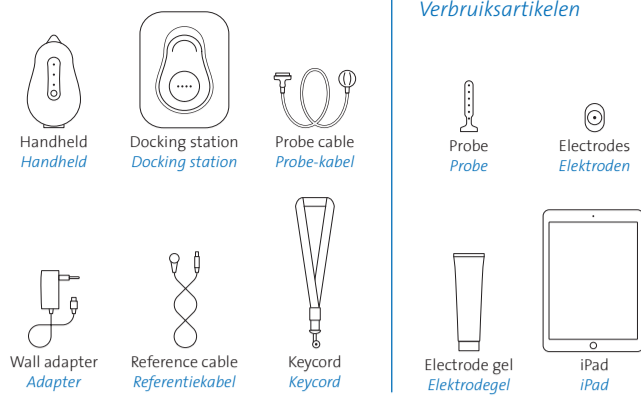
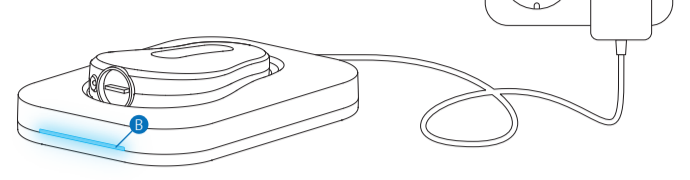
### Consumables Verbruiksartikelen

### 2 Charge the handheld Laad de handheld op

- A** Place the docking station at least 1.5 meters away from the treatment area. Plug the wall adapter into the docking station and into a wall socket.
- B** Place the handheld on the docking station. The docking station will blink blue, the handheld is charging.

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### 3 Download the MAPLe app Download de MAPLe app

- A** Go to the App store on your iPad and search for 'maple app'. Download and install the MAPLeApp.
- B** Connect the iPad to the Wi-Fi network belonging to the MAPLe system. Go to your iPad's 'Settings', select 'Wi-Fi' and select the appropriate Wi-Fi network starting with MPFT. Enter the password supplied with your system and open the MAPLe app.
- C** Create an administrator account and password. The first user created is automatically the administrator with accompanying username 'admin'. Quit the MAPLe app.

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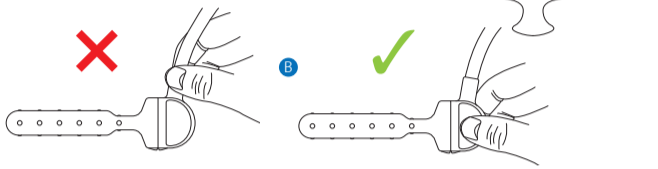
### 4 Connect probe cable and probe Verbind de probe-kabel en probe

- A** Align the probe cable and the probe at the alignment marks, connect the parts.
- B** Hold the probe at the appropriate grip. Do not touch the cable during treatment, this may influence the measurement.

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
- A** Align the probe cable with the handheld at the alignment marks, connect the parts.
- B** Connect the reference cable with the handheld.

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
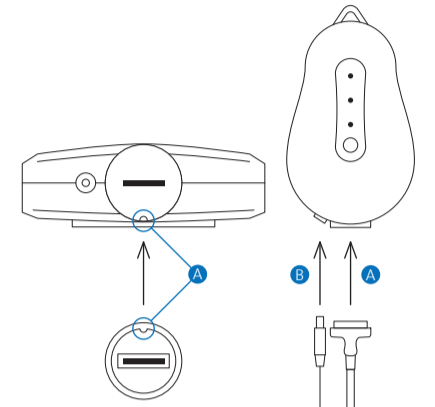
- A** Press the handheld/off switch for 2 seconds until the 3 LEDs light up.
- B** The electromyography (EMG) and electro stimulation (ES) LEDs will blink until the handheld is connected.
- C** Open the MAPLeApp when the LEDs have stopped blinking.

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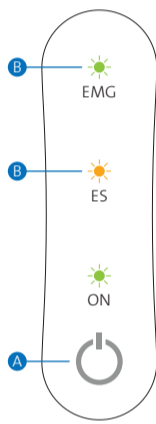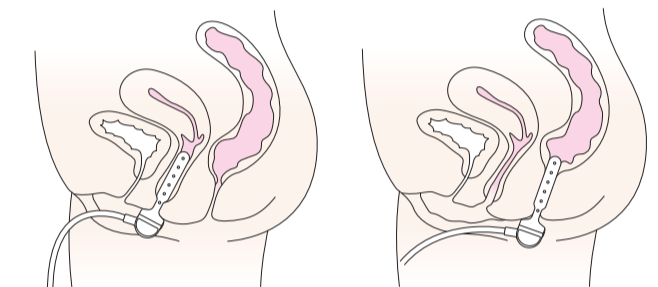
- A** Apply electrode gel before placement of the probe.
- B** Insert the probe vaginally/anaally until the bottom electrodes lay inside the body orifice. The cable shall point to the nose/belly of the patient.

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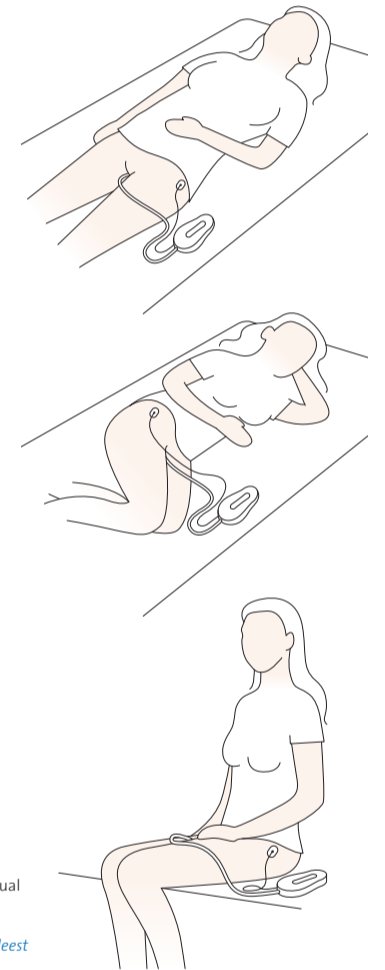Treatment may be provided in supine position, lying on one side, sitting or standing. Please make sure to place the reference electrode on the patient's hipbone before starting treatment.

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
Please ensure to read the instruction manual before using the MAPLe system.

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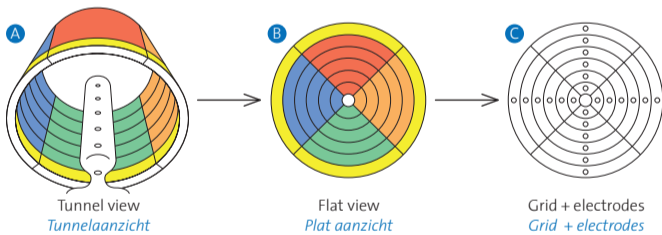
- A** Tunnel view. The grid represents a tunnel view of the probe.
- B** Flat view. The outer yellow circle represents the lowest ring of the probe electrodes, nearest to either the Urethral sphincter or External Anal sphincter.
- C** The grid is divided per probe electrode.

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
- D** Operator view; operator is looking at patient's front. The right side of the grid corresponds to the left side of the patient. The left side of the grid corresponds to the right side of the patient.
- E** Patient view; patient is looking at iPad while being treated. The right side of the grid corresponds to the right side of the patient. The left side of the grid corresponds to the left side of the patient.

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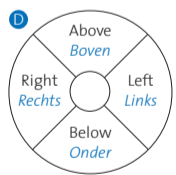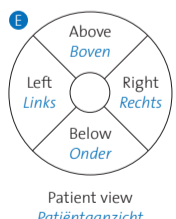
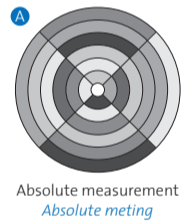
- A** Absolute measurement. Measurements are displayed in uV's (microvolts) in a grey-scale.
- B** Reference comparison mode. Measurements are compared to data from a reference group and are displayed as over activity (red), under activity (blue) or same as reference group (white). More intense colors represent a larger deviation with the reference group. To select this option go to 'Measurement', 'measurement settings', 'compare with reference' and select one of the possibilities.

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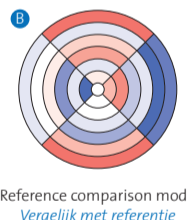
### 11 Cleaning the probe Probe schoonmaken

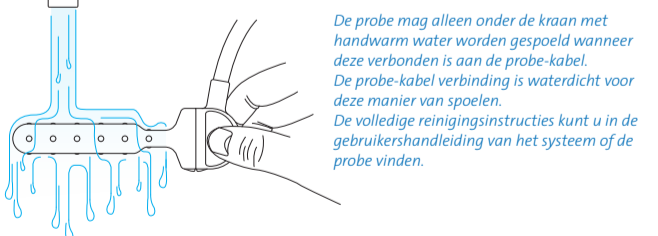The probe may be rinsed under a tap with lukewarm water only while connected to the probe cable. The probe-probe cable connection is waterproof for this rinsing method. Please see the system user manual or the probe user manual for full cleaning instructions.

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Did you know that the MAPLe system does not encounter co-contractions? Wist u dat het MAPLe systeem geen co-contracties meet?

**⚠ Safety warnings**  
- Read the instruction manual before using the product.  
- Only use official consumables provided by the manufacturer.

Novuqare Pelvic Health B.V.  
All rights reserved.  
QG\_MAPLe\_EN-NL-ES-PT\_v6

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